Mamaco[®]

KASUTUSJUHEND

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INSTRUCTIONS FOR USE

Mamaco® soft belly wrap for use during pregnancy, postpartum and after abdominal surgery

I class medical device

The instructions for use are applicable to the following Mamaco® medical belly wraps:

G0121-S, Mamaco® soft belly wrap with gel bag, S
G0121-M, Mamaco® soft belly wrap with gel bag, M
G0121-L, Mamaco® soft belly wrap with gel bag, L
G0121-XL, Mamaco® soft belly wrap with gel bag, XL
B0221-S, Mamaco® soft belly wrap with pelvic belt, S
B0221-M, Mamaco® soft belly wrap with pelvic belt, M
B0221-L, Mamaco® soft belly wrap with pelvic belt, L
B0221-XL, Mamaco® soft belly wrap with pelvic belt, XL

The product package contains:

1 belly wrap (S, M, L, XL)
1 pelvic belt or gel bag with support belt
Instructions for use

Device sterility:

Mamaco® medical belly wrap does not qualify as a sterile medical device and is not packaged or shipped sterile.

Intended purpose:

Mamaco® soft belly wrap for pregnancy and early postpartum recovery. Designed to prevent and alleviate pregnancy-related complaints. Offers support for abdominal area, including after surgeries (e.g. C-section, hysterectomy, corrective abdominal surgery, Diastasis Recti surgery) for faster and easier recovery. Best fit when third trimester pregnant and 4–8 weeks postpartum. After abdominoplasty, start using the belly wrap after 24 hours until needed or full recovery. The belly wrap is suitable for use from morning to evening with or without a belt, based on how you feel. Consult a healthcare professional or manufacturer if necessary.

User group:

The belly wrap is designed for women to use during pregnancy, postpartum or after abdominal surgery.

Indications:

- During pregnancy: pain syndromes in the lower back, sacrum, pubic and pelvic areas, abdominal wall pain; prevention of stretch marks.
- Postpartum: elongated anterior stomach and abdominal muscles (weak support for the abdominal organs, diastasis), pain syndromes in lower back, sacrum, pubic and pelvic areas.
- Swelling and pain in the wound area after abdominal surgery (e.g. caesarean section, hysterectomy, diastasis corrective abdominal surgery).

MATERIALS

Fabric: bamboo viscose or cotton (depending on model) 93%, elastane 7%

Belt: 65% polyester, 35% latex

Hook and loop fasteners: polypropylene

Gel bag (model with gel bag):
water, propylene glycol, phthalate-free PVC

MAINTENANCE INSTRUCTIONS

- Remove the gel bag (model with gel bag) from the belly wrap before washing.
- Attach the snap fasteners and use a laundry bag for washing.
- Wash with similar colours, using a gentle wash program at a water temperature up to 40 °C.
- Ironing is allowed at the average temperature of 110...150 °C. Ironing of the belt is not required.
- Do not bleach.
- Dry cleaning is permitted in accordance with the symbol.
- Do not use a tumble dryer.
- After washing, place the belly wrap on a flat surface to dry.
- The same washing conditions apply to the belt as to the belly wrap.
- If necessary, wash the gel bag with warm water and soap, dry with a soft towel.

NB! When worn, the pilling of fabric may occur. This is a natural feature of the fabric that does not reduce the properties of the belly wrap. The pills can be removed with a fabric shaver.

STORAGE AND DISPOSAL

Mamaco® soft belly wraps have a shelf life of up to 2 years (the date indicated on the package). It is recommended to store the device in a clean place in the original packaging at a temperature between –10 °C and 40 °C with a relative humidity of less than 80%. Keep away from direct sunlight, high temperatures, and chemicals. The gel bag (model with gel bag) is not usable after rupture. After use, the device (including all parts) is suitable for household waste.

WARNINGS AND PRECAUTIONS

1. This is a medical device. The product is not a substitute for medical care. Consult a healthcare professional who is aware of your condition before using the product.

2. The belly wrap is intended for use in accordance with the instructions of use. If your condition deteriorates during wearing the belly wrap, remove the device and contact a healthcare professional if necessary:

2.1 if you wear a belly wrap or belt around your body for a long time too tight, you may feel dizzy, faint, feel sick and increased heart rate.

2.2 the use of a belly wrap before the recommended time of pregnancy (during the first 3 months of pregnancy) is not recommended due to an overall increase in abortion rates during this period.

2.3 people who are allergic to latex may experience an allergic reaction when touching or placing the support belt on their skin.

2.4 heating the gel bag (model with gel bag) to extremely high temperatures (e.g. taken directly from hot water) or low temperatures (taken directly from the freezer) and then placing it against the skin (including without the wrapper) may cause skin damage. Use the prescribed pocket or make sure that there is a thin cloth between the gel bag and the skin.

2.5 dropping the gel bag, applying excessive pressure to it or touching it with a sharp object may cause the gel bag to break, rendering the gel bag unusable.

2.5.1 if the gel bag packaging breaks, rinse the liquid off the skin and surfaces, in case of contact with eyes or if swallowed, rinse with water and inhale fresh air after inhalation. Consult a doctor if symptoms occur. Dispose the empty packaging as household waste. Keep out of the reach of children.

3. Prolonged regular use (or if the size is too small) may wear out the product in such a way as to cause the seams or material to tear, the fasteners to tear at the seam. The lifespan and functionality of the device may therefore be reduced: the belly wrap may not provide the expected result or support.

4. Failure to follow the maintenance or storage instructions. Leaving the product in the sun for too long or using in extreme temperatures (e.g. ironing, washing, etc.) may affect the life of the product and may reduce its functionality: the belly wrap may not provide the expected results. The fabric has a recommended ironing and washing temperature range, and exceeding the recommended limits may damage the fabric.

5. Placing the belly wrap in a wrong place or incorrectly does not cause damage to health, but does not have the expected medical effect.

6. The belly wrap includes a gel bag with a support belt or a pelvic belt (depending on the model). The manufacturer is not responsible if the user selects components from other manufacturers for Mamaco® products.

7. All incidents involving the device must be reported to the Manufacturer and to the competent authority of the Member State in which the user is established (Health Authority).

EXPLANATION OF SYMBOLS

(marking on packages and instructions for use)

Manufacturer

See the user guide

Expiration date

Delicate package

Batch number

Medical device

Contains latex

Conformity marking. The product complies with all relevant regulations, in particular health, safety and environmental protection requirements. The product can be marketed and sold in European Union.

Instructions for use vers.1.1 (1st February 2022) is published in the following languages: Estonian, English

Manufacturer: Fysiomed OÜ

Trade mark: Mamaco®

Address: Pulga tee 23, Reiu küla, Häädemeeeste vald, Pärnumaa, 86508, Estonia. Phone +372 510 9561

INSTALLATION INSTRUCTIONS

(both during pregnancy and after childbirth)

Model B0221 soft belly wrap with pelvic belt and Model G0121 soft belly wrap with gel bag

Choosing the right size of a belly wrap and the right application technique ensures the desired wearing comfort and function. There is a size chart on the pack and instructions for choosing the correct size. The best position for putting on the belly wrap is lying on the back, but it can also be put on while standing. It is important to make sure that the belly wrap stays around the body with tension (layers on top of each other), while the tension/compression must be pleasant to the wearer. You can adjust the belly wrap to the right tension by trying it out.

1. Lying on your back, place the belly wrap on your stomach, preferably in the morning before getting out of bed. Make sure the branch of the belly wrap is on the left, the logo is visible and is in the middle below the back. The belly wrap is applied to the naked body, so that it remains under the clothes.

2. Bring the straight end of the belly wrap with your right or left hand to the left hip (down or behind your back after delivery). The lower edge of the belly wrap must cover the hip knuckles. If necessary, lift your hips off the ground and use both hands to move the belly wrap to the correct distance and height and to keep the upper layer lightly tense. In the case of a caesarean section, the belly wrap should cover the wound area and be about 3–4 cm below the sutures. The pressure on the sutures should be pleasantly supportive, even throughout the swelling area, and should not cause an uncomfortable feeling of tightness or grooving.

3. Attach the lower branch first, then the upper. The belly wrap branches allow the pressure on the upper and lower abdomen to be adjusted differently. If necessary, you can also adjust the bandage so that it is comfortable around the abdomen and the lower part of the fabric does not remain wrinkled and the lower strap attachments are positioned exactly on the hip knuckles.

4. Finally, attach the support belt (see below). In the case of a caesarean section, make sure that the belt stays on the seams and does not press on the seams from above or below.

4a. - 4c. **Model B0221 soft belly wrap with pelvic belt.** Place the pelvic belt in the middle of the lower back. Attach the belt to the belly wrap by gently stretching it with the inner hook and loop fasteners and then fasten it around the body with the hook and loop fasteners (a). You can give stronger support to your back and pelvic area by tightening the upper pelvic belt branches (b). The pelvic belt can also be used the other way around, i.e. by placing the belt on the lower abdomen (attaching it to the belly wrap) and then attaching it behind the back, in which case the upper branches remain to provide additional support for the lower abdomen (c).

4d. - 4f. **Model G0121 soft belly wrap with gel bag.** Slightly tighten the support belt by stretching it with one or both hands and then attaching it to the hip hook and loop fasteners (d). The support belt is intended for use on both the lower abdomen (e) and the lower back (f) as desired/comfortable. Inside the belt is a pocket for a gel bag for hot-cold treatments.